

PureRegen™ is a patented professional low-frequency RF device with a temperature-controlled safety system to tighten skin.

CONTROLLED PROFESSIONAL

SKIN TIGHTENING SYSTEM



PureRegen™ is a patented professional low-frequency RF device with a temperature-controlled safety system to tighten facial skin. It can also be used to tighten skin on other parts of the body after a session with the PureLight™ device.

PureRegen™ is designed for professional use. The unit is compact, and is safe and easy to operate.




The shape of the electrodes makes it easy to apply just the right pressure to the skin, even in curved areas.




Low resistance between the electrodes and the skin means current can be concentrated where it is needed.



This means the area can be treated effectively without harming the skin.

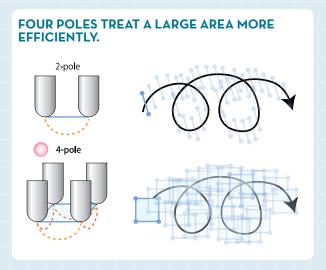

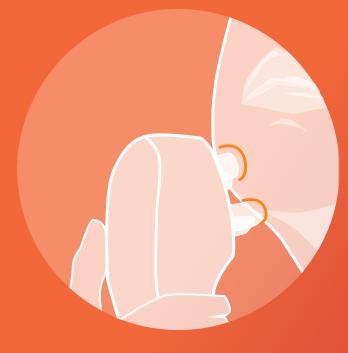

Benefits

- Allows for release of fat and cellulite from adipose tissue
- Encourages regeneration of collagen to reduce wrinkles and tighten skin
- Reduces the number of visible pores
- Reduces the coloring & effects of skin pigmentation
- 5 Improves moisture and viscoelasticity of skin



Skin Rejuvenation & Tightening Study

This study was conducted by Inforward, Inc., an independent agency, over a four-week period to measure the efficacy of PureRegen™ in skin rejuvenation and tightening. The subjects' entire face was treated with PureRegen™ in three 10-minute sessions a week for four weeks. Visible pores, facial pigmentation, wrinkles, moisture content, and

viscoelasticity were measured before and after the use of PureRegen™. The results of the study show marked reduction in the number of visible pores, reduction in the pigmentation area, reduction in width and depth of wrinkles, as well as an increase in moisture and viscoelasticity of facial skin.

EVALUATION OF VISIBLE PORES

AFTER 4 WEEKS



1505 pcs

981 pcs

EVALUATION OF PIGMENTATION

AFTER 4 WEEKS




Before 62 pcs / 362 mm²

After 43 pcs / 240 mm²


EVALUATION OF THE NUMBER OF WRINKLES

AFTER 15 MINUTES USE


Width of wrinkes Depth of wrinkes

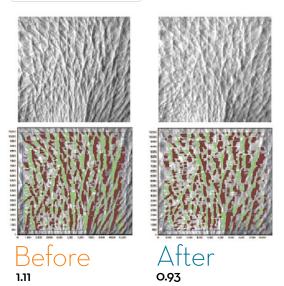

EVALUATION OF THE STRATUM CORNEUM WATER CONTENT

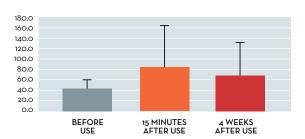

EVALUATION OF SKIN VISCOELASTICITY

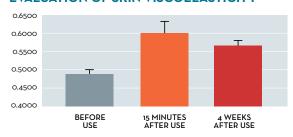





PROVEN & PATENTED TEMPERATURE-CONTROLLED PROFESSIONAL SKIN TIGHTENING SYSTEM

SPECIFICATIONS

Rated Voltage and Frequency	AC100V-240V 50/60Hz
Rated Input	40W
Adapter Model	DC15V 3A
Output Frequency	1 MHz
Timer	20 minutes
Safety Device	OVERCURRENT PROTECTION, OVERTEMPERATURE PROTECTION
Body Dimensions	W335mm x D150mm x H65mm
Weight	About 624g



Xtreem Pulse LLC

an integrative healthcare solutions company 353 West 29th Street, Suite 3 New York, NY 10001 U.S. +1 212 643 9062 info@xtreempulse.com